


THE ADVANCED WORKHORSE

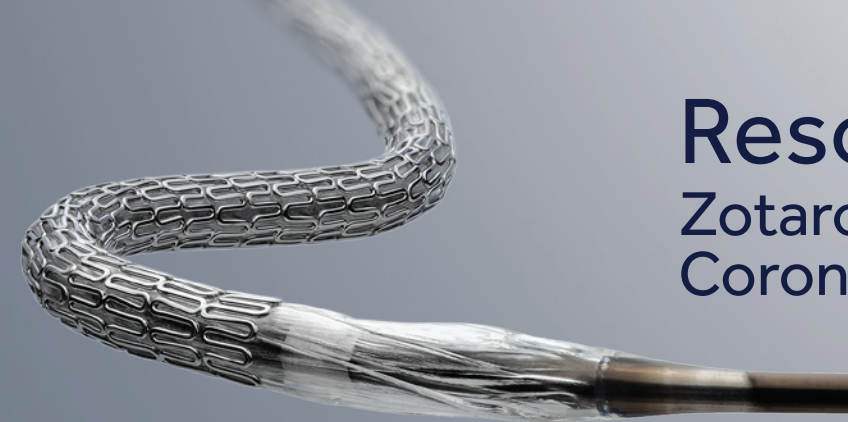

Resolute Onyx™
Zotarolimus-Eluting
Coronary Stent System

Compliance Data

Pressure kPa (atm)	Stent Diameter Deployed Stent I.D. (mm)								
	2.00	2.25	2.50	2.75	3.00	3.50	4.00	4.50	5.00
709 (7)	1.85	2.05	2.25	2.45	2.75	3.05	3.60	4.10	4.55
811 (8)	1.90	2.10	2.30	2.55	2.80	3.15	3.70	4.20	4.65
912 (9)	1.90	2.15	2.35	2.60	2.90	3.25	3.80	4.30	4.80
1013 (10)	1.95	2.20	2.45	2.65	2.95	3.35	3.85	4.40	4.90
1115 (11)	2.00	2.25	2.50	2.70	3.00	3.40	3.95	4.45	4.95
1216 (12)	2.05	2.30	2.55	2.75	3.05	3.45	4.00	4.50	5.05
1317 (13)	2.05	2.35	2.55	2.80	3.10	3.50	4.05	4.55	5.10
1419 (14)	2.10	2.35	2.60	2.80	3.10	3.55	4.05	4.60	5.15
1520 (15)	2.10	2.35	2.60	2.85	3.15	3.55	4.10	4.65	5.20
1621 (16)	2.15	2.40	2.65	2.90	3.20	3.60	4.15	4.70	5.25
1723 (17)	2.15	2.40	2.70	2.90	3.20	3.65	4.20	4.80	5.30
1824 (18)	2.20	2.45	2.70	2.95	3.25	3.70	4.25	4.85	5.35
1925 (19)	2.20	2.45	2.75	3.00	3.30	3.75	4.30	—	—
2027 (20)	2.25	2.50	2.75	3.00	3.35	3.80	4.35	—	—
2128 (21)	2.25	2.50	2.80	3.05	3.40	3.80	4.40	—	—
MSID	3.25 ¹	3.25 ¹	3.25 ¹	3.75 ¹	3.75 ¹	4.75 ¹	4.75 ¹	5.75 ¹	5.75 ¹

Nominal pressure

Rated burst pressure²

Maximum stent I.D.

¹Do not postdilate greater than listed value.

²Do not exceed rated burst pressure.

Europe
Medtronic Intl. Trading SARL
Route du Molliou 31
Case Postale
CH-1131 Tolochenaz
Switzerland
Tel: 41.21.802.7000
Fax: 41.21.802.7900

Asia Pacific
Medtronic Intl. Ltd.
49 Changi South Avenue 2
Nasaco Tech Centre
Singapore 486056
Singapore
Tel: 65.6436.5000
Fax: 65.6776.6335

Latin America
Medtronic
9850 Doral Blvd., 4th Floor
Miami, FL 33178
USA
Tel: 786.709.4200
Fax: 786.709.4244

© 2017 Medtronic. All rights reserved. Medtronic, Medtronic logo and Further, Together are trademarks of Medtronic. All other brands are trademarks of a Medtronic company. For distribution only in markets where the Resolute Onyx™ coronary stent has been approved. Not for distribution in USA, Japan or France. UC201705543ML 2/17

medtronic.com

Medtronic
Further, Together